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INDEX

PRESENTATION OF MANUAL	23
IDENTIFICATION OF EQUIPMENT	24
- Dear customer	24
- Identification	24
- Description of Equipment	25
- Principles and fundamentals applied to the product operation	25
- Indication of Equipment	25
MODULES, ACCESSORIES, OPTIONS AND MATERIALS OF CONSUMPTION	26
TECHNICAL SPECIFICATIONS	27
- General	27
- Electrical	27
- Weight	27
- Other specifications	28
- Norms employed	28
- Dimensions	29
- Packing symbols	30
- Product symbols	30
- Content of accessible and non-accessible demarcations	31
INSTALLATION OF EQUIPMENT	31
OPERATION OF EQUIPMENT	32
- Powering the Compressor	32
- Compressed air outlet	33
- Drainage	33
PRECAUTIONS, RESTRICTIONS AND WARNINGS	34
- Transportation, storage and operation	34
- Precautions and warnings "during the installation" of equipment	34
- Recommendations for the equipment maintenance	35
- Precautions and warnings "during the use" of equipment	35
- Precautions and warnings "after" the use of equipment	36
- Precautions and warnings during the "cleaning" of equipment	36
- Sensitivity to environmental conditions in normal situations of use	36
- Precautions in case of alteration in the functioning of equipment	36
- Precautions to be adopted against foreseeable or uncommon risks, related to the deactivation and abandoning of equipment	36
- Additional procedures for reuse	36
CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION	37
- Preventive Maintenance	37
- Corrective Maintenance	37
- Cleaning	38
UNFORESEEN EVENTS – SOLUTION OF PROBLEMS	38
WARRANTY OF EQUIPMENT	40
FINAL CONSIDERATIONS	40

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ATTENTION

For greater safety:

Read and understand all the directions contained in this manual before installing or operating this equipment.

Note: This manual must be read by all the operators of the equipment.

IDENTIFICATION OF THE EQUIPMENT

Dear Customer

Congratulations. You have made an excellent choice by purchasing a Dabi Atlante quality product, among the most advanced and innovative in the world. This manual provides a general overview of your product and provides important details to help you solve potential problems. Please read this manual and keep it for your reference.

Identification

Technical Name: Air compressor

Brand: Dabi Atlante

IDENTIFICATION OF THE EQUIPMENT

Description of Equipment

The compressors have a modern and compact design. Super quiet, it maintains a pleasant work environment and ensures the calmness of the patient during the procedure execution.

Designed to provide compressed air for clinical and laboratory use, it features stable performance, high flow capacity, low energy consumption and is free from oil or smoke emission, vapors or unpleasant odors.

Equipped with a pressure gauge (manometer) to measure the tank pressure, manometer for outlet pressure, outlet pressure regulator with air filter and water drain, register to control the outflow rate and register to drain the accumulated water in the tank, features which facilitate the equipment's operation and maintenance.

Safety System with valve that starts running for pressure release if there is any failure from the pressure switch and overload shield in order to protect the equipment from overheating.

Models with tanks of 40 and 65 liters both with internal antioxidant painting, ensuring greater durability of the compressor.

Principles and fundamentals applied to the product operation

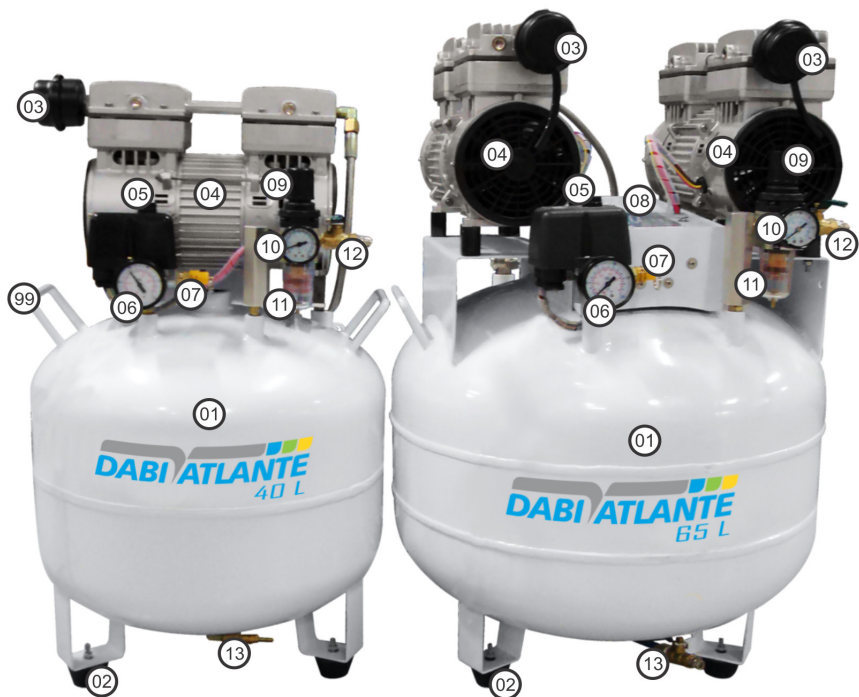
All components of the equipment are mounted over the air tank. One or more electric engines trigger the pistons' system with rings made of non-metallic material resistant to abrasion and high temperatures that are able to operate without lubrication for a long period of time. The air is compressed within the tank and a manometer displays its pressure. A check valve, drain register, safety valve and solenoid valve are also installed.

Indication of equipment

This equipment is exclusively for exams, and must be used and handled by a trained person (duly accredited professional, as per the country's local legislation), in compliance with the instruction of this manual.

It is mandatory to use the equipment in perfect conditions only and to protect the operator him or herself, the patient, and any other person against any eventual risks.

MODULES, ACCESSORIES, OPTIONS AND MATERIALS OF CONSUMPTION

Compressor 40L

Compressor 65L

- 01 - Air tank
- 02 - Rubber feet
- 03 - Air intake filter
- 04 - Electric engine
- 05 - On - off switch
- 06 - Inner pressure gauge/manometer
- 07 - Safety valve
- 08 - Overload protector
- 09 - Register of the outlet pressure
- 10 - Manometer of the outlet pressure
- 11 - Output filter and tank
- 12 - Outflow rate register
- 13 - Drainage register

TECHNICAL SPECIFICATIONS

General

Models
Compressor 40L Compressor 65L
Classification of Equipment as per standard IEC 60601-1:
Protection against Electric Shock - Type BF and Class II Equipment (IEC 60601-1)
Protection against harmful water penetration
IPX 0
Mode of operation
Continuous operation.

Electrical

Supply voltage and frequency
220V~ ±10% / 60Hz or 220V~ ±10% / 50Hz or 127V~ ±10% / 60Hz
Rated current
Compressor 40L 127V~: 7,3 A Compressor 40L 220V~: 5 A Compressor 65L 127V~: 14,6 A Compressor 65L 220V~: 10 A
General power and energy consumption
Compressor 40L 127V~: 830 VA / 0,83 KW/h Compressor 40L 220V~: 1100 VA / 1,1 KW/h Compressor 65L 127V~: 1660 VA / 1,66 KW/h Compressor 65L 220V~: 2200 VA / 2,2 KW/h
Engine power
Compressor 40L 127V~: 1 engine de 1,12 HP Compressor 40L 220V~: 1 engine de 1,14 HP Compressor 65L 127V~: 2 engines de 1,12 HP Compressor 65L 220V~: 2 engines de 1,14 HP

Weight

Gross weight
Compressor 40L: 45,8 Kg Compressor 65L: 71,2 Kg
Net weight
Compressor 40L: 30,6 Kg Compressor 65L: 50,4 Kg

English

TECHNICAL SPECIFICATIONS

Other specifications

Capacity of dental sets

Compressor 40L: 1 dental sets

Compressor 65L: 2 dental sets

Air Tank capacity

Compressor 40L: 40 liters

Compressor 65L: 65 liters

Noise level (700mm from equipment)

Compressor 40L: 65 db

Compressor 65L: 71 db

Minimum and maximum pressure

Compressor 40L: 5,71 Kg/cm² e 8,16 Kg / cm²

Compressor 65L: 5,71 Kg/cm² e 8,16 Kg / cm²

Air displacement

Compressor 40L: 212 l/min

Compressor 65L: 424 l/min

Norms employed:

This product according to the following standards:

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NBR ISO 14971:2009- Medical devices - application of risk management medical devices;

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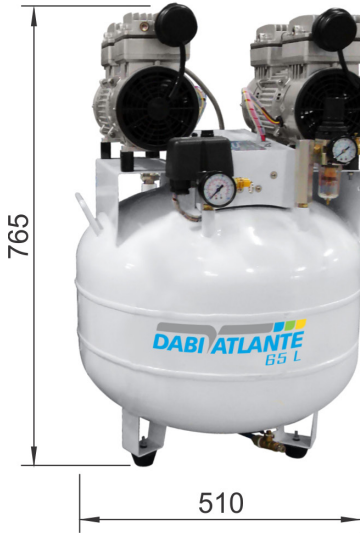
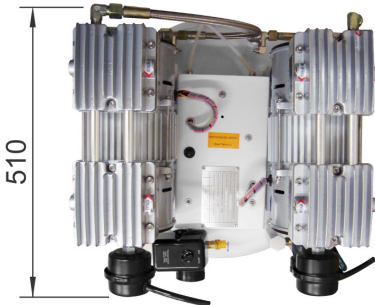
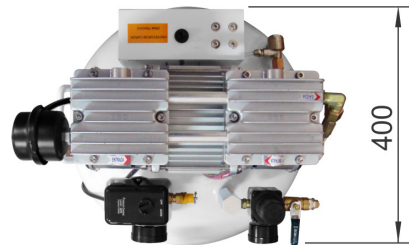
TECHNICAL SPECIFICATIONS

Packing symbols

Maximum stack, determines the maximum amount of boxes that can be stacked during transport and storage "as packaging".

It determines that the equipment must be stored or transported with the arrow indication up.

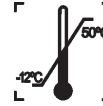
It determines that the equipment must be stored and transported with care (no falls and impacts).

It determines that the equipment must be stored or transported with moisture protection (do not expose to rain, water splashing or ground).

It determines that the equipment must be stored or transported with light shield.

It determines the temperature limits within the packaging must be stored or transported.

Product symbols

Careful: It indicates an important instruction for the operation of the product. Not following it can cause dangerous malfunctioning.

Note: It indicates useful information for operation of the product.

Important: It indicates an instruction of safety for operation of the product. Not following it, can lead to serious danger to the patient.

Warning - see the manual.

Turned on position.

Turned off position.

B type equipment

Grounding (at several points of the equipment) indicates the condition of being grounded.

TECHNICAL SPECIFICATIONS

Content of accessible and non-accessible demarcations

AV. PRES. CASTELO BRANCO, 2525
RIBEIRÃO PRETO - SP
+55 16 3512-1212 - CEP:14095-000

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English

INSTALLATION OF EQUIPMENT

The installation of this equipment requires specialized technical assistance (Dabi Atlante).

These information also make part of the Manual of Installation and Maintenance of the equipment that can be found with the authorized Dabi Atlante technician.

- This equipment shall only be able to be unpacked and installed by a Dabi Atlante authorized technician, under penalty of losing the warranty, as only (s)he has the information, suitable tools and training required to execute this task.

- Dabi Atlante bears no responsibility for damages or accidents caused by poor installation executed by a technician not authorized by Dabi Atlante.

- Only after the equipment has been installed and duly tested by the authorized technician representing Dabi Atlante, will it be ready to start work operations.

INSTALLATION OF EQUIPMENT

Compressor 40L

Voltage	Rated current	Circuit breaker
127V~ ±10%	7,3 A	10 A
220V~ ±10%	5 A	10 A

Compressor 65L

Voltage	Rated current	Circuit breaker
127V~ ±10%	14,6 A	20 A
220V~ ±10%	10 A	16 A

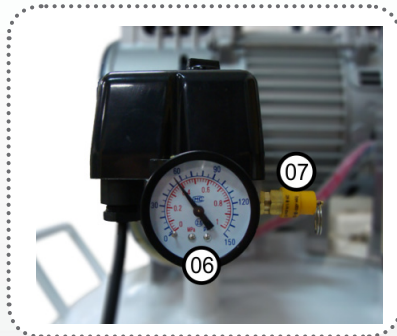
Wiring Diameter (tread): For distance up to 5m from the circuit breaker until equipment socket, wiring should be 2,5 mm² section; from 5 to 15m, 4mm wiring; from 15 to 50m, 6mm wiring (According to standard ABNT NBR-0148, metric series PVC 70 °C).

OPERATION OF EQUIPMENT

Powering the Compressor

Turn the power switch (05) to position "AUTO". The compressor will be turned on until the inner pressure of the tank displayed on the gauge (06) reaches 8.16 kgf / cm² (0.8 MPa - 116 psi), on this value the pressure switch will turn off the compressor which will be powered again when the inner pressure reaches of 5.71 Kgf/cm² (0.56 MPa - 81,22 psi), and so successively and automatically.

If the pressure switch fails, the safety valve (07) will start running by releasing the excessive pressure.


OPERATION OF EQUIPMENT

Compressed air outlet

The air pressure on the outlet can be adjusted through the register (09), check the setting on the pressure gauge (10). The air outflow rate can also be adjusted in the register (12).


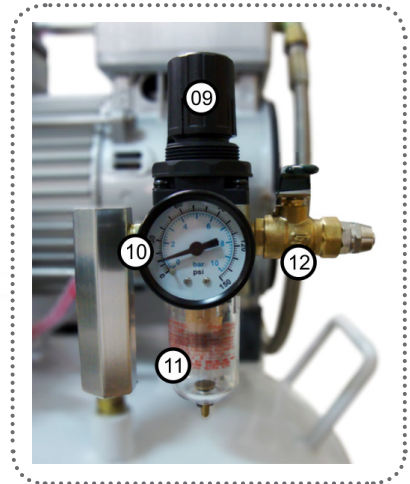
In tandem with the pressure and flow rate registers, there is a filter that removes moisture and impurities remaining in the air, the moisture that is removed from the outlet will get stored in the reservoir (11), to empty it press the pin underneath the reservoir.

Drainage

Drainage of the air tank should be performed to eliminate the condensed water accumulation inside the tank. The drainage frequency depends on environmental conditions and operating time, on average the drainage is recommended to be done on a daily basis.

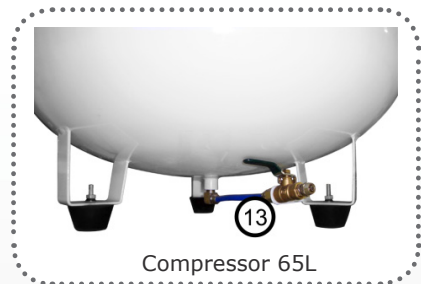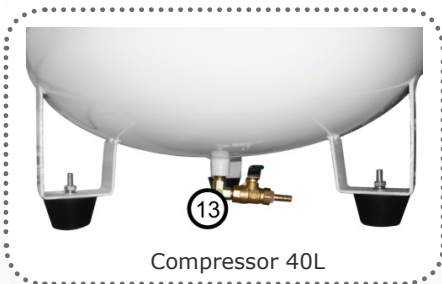To perform the drainage, insert the hose which is connected to the register (13) located underneath the tank inside a container for water disposal. Open the register slowly until all the accumulated water is drained and air starts to flow again. Close the register firmly so that no leakage can occur.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Transportation, storage and operation

This equipment must be transported and stored observing the following directions:

- Avoid falls and impacts;
- Keep it dry, do not expose it to rain, water drops or wet floor;
- Keep it away from water and direct sunlight, and in its original wrapping;
- Don't move it over irregular surfaces, protect it from rain and observe the maximum stack quantity specified in the packaging;
- Transportation and storage temperature range: -12°C to 50°C.
- Ambient temperature range recommended by Dabi Atlante +10 ° C to +35 ° C.

The Equipment maintains its condition of safety and efficacy, provided that it is maintained (stored) as mentioned in this instruction of use. Thus, the equipment will not lose or alter its physical and dimensional features.

Before transporting the compressor, all air from the tank must be exhausted.

Precautions and warnings “during the installation” of equipment

- The equipment should only be installed by Dabi Atlante authorized technical assistance or technicians.

- Check that the socket in which the device will be connected has a ground connection. According to the ABNT standard, this is essential for the safe operation of the system;

- Position the unit in a place where it will not get wet.

- Install the unit in a place where it will not be damaged by the pressure, temperature, humidity, direct sunlight, dust, salts.

- The unit should not be submitted to inclination, excessive vibrations, or blows (including during transportation and handling).

- This equipment was not planned for use in an environment where vapors, anesthetic mixtures inflammable with air, or oxygen and nitrous oxide can be detected.

- Check the voltage of the equipment at the moment of executing the electrical installation.

- The equipment must be grounded correctly.

- Use only hoses and couplings of high pressure specifically designed to compressors, it is very important for your safety and of your equipment. If you have questions, see the Dabi Atlante Technical Assistance Network.

These information also make part of the Manual of Installation and Maintenance of the equipment that can be found with the authorized Dabi Atlante technician.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Recommendations for the equipment maintenance

Your Dabi Atlante equipment has been designed and developed according to the standards of modern technology. Similarly to other kinds of equipment, it requires special care, which is many times neglected due to several reasons and circumstances. Therefore, here are some important reminders for your daily routine. Try to follow these simple rules, which will save you a lot of time and will avoid unnecessary expenses once they start making part of your working procedure.

Precautions and warnings “during the use” of equipment

- The equipment should only be operated by duly enabled and trained technicians
- If any maintenance should be required, only use services of the Dabi Atlante Authorized Technical Assistance.
- Although this equipment has been planned in accordance with the standards of electromagnetic compatibility, it can, in very extreme conditions, cause interference with other equipment. Do not use this equipment together with other devices very sensitive to interference or with devices which create high electromagnetic disturbance.
- Avoid spilling water or other liquids inside the equipment, which could cause short circuits.
- Do not allow patients and especially children get close to the compressor.
- Dabi Atlante air compressor is designed for use according to its air flow capacity, while using it, does not exceed their technical specifications. Dabi Atlante assumes no liability for damages/accidents resulting from improper use or non-compliance with the instructions described in this manual.
- Do not touch the compressor head (the upper cooling part) during operation, even after the compressor is turned off, those parts still remain hot for some time, so avoid contact.
- Make sure periodically that the air inlet and outlet are not obstructed.
- If there is power outage, it is recommended to turn off the compressor power switch to prevent the disarming of the thermal relay when power returns again.
- Before disconnecting the pressure hose from the compressor for any reason, first close the air outlet register and release the residual pressure through the drainage register (13).
- There is condensation from air humidity inside the tank that must be removed periodically, we recommend the removal of this water accumulation to be carried out daily.

NOTE: We recommend reading this manual to the full understanding of it. Use it as a resource of information.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Dabi Atlante shall not be responsible for:

- Use of the equipment differing from that for which it is intended.
- Damages caused to the equipment, the professional and/or the patient by the incorrect installation and erroneous procedures of maintenance, differing from those described in these Instructions for use which come with the equipment or by the incorrect operation of it.

Precautions and warnings "after" the use of equipment

- Turn off the main switch of the chair when it is not in use for an extended period of time.
- Always maintain the equipment clean for the next operation.
- Do not modify any part of the equipment. Do not disconnect the cable or other connections without need.

Precautions and warnings during the "cleaning" of equipment

- Before cleaning the equipment, turn off the main switch.
- Avoid spilling water, even accidentally, or other liquids inside the equipment, which could cause short circuits.
- Do not use microabrasive material or steel wool when cleaning, or employ organic solvents or detergents which contain solvents such as ether, stain remover etc.

Sensitivity to environmental conditions in normal situations of use

The equipment has been planned not to be sensitive to interference such as magnetic fields, external electrical factors, electrostatic discharge, pressure or variance of pressure, provided that the equipment is installed, maintained, clean, preserved, transported and operated as per this instruction for use.

Precautions in case of alteration in the functioning of equipment

If the equipment has any abnormality, check if the problem is related to any item listed in the topic of unforeseen events (failures, causes and solutions). If it is not possible to resolve the problem, turn off the equipment, remove the power supply cable from the socket and contact your representative (Dabi Atlante).

Precautions to be adopted against foreseeable or uncommon risks, related to the deactivation and abandoning of equipment

In order to avoid environmental contamination or undue use of the Equipment after it has become useless, it should be discarded in the suitable place (as per the local legislation of the country).

- Pay attention to the local legislation of the country for the conditions of installation and disposal of residue.

Additional procedures for reuse

The equipment can be reused in undetermined, i.e. unlimited, quantities, only needing to be cleaned and disinfected.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

Preventive Maintenance

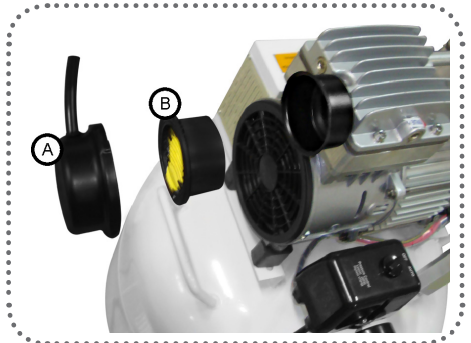
The equipment should be calibrated routinely, as per the legislation in force in the country. But never with a period exceeding 3 years.

In order to protect your equipment, seek Dabi Atlante technical assistance for periodic revisions of preventive maintenance.

Filter Change:

Keep the air filter(s) (B) clean and in good condition of usage. The air filter (B) prevents dust from entering into the compressor and reduces the operating noise. Over time, the filter will become dirty, reducing the air inlet flow and causing the decrease of compressor performance. The filter should be replaced every 3 months on average or before, depending on the air quality. Additional filters are available for purchase through the Dabi Atlante Technical Assistance network.

To perform the change, open the filter enclosure by turning the metal cover (A); remove the old filter and insert the new one; put the cover (A) again by turning it in the opposite direction.



Corrective Maintenance

If the equipment has any abnormality, check if the problem is related to any of the items listed in the item Unforeseen Events (situation, cause and solution). If it is not possible to solve the problem, turn off the equipment, and request Dabi Atlante technical assistance.

Overload protector:

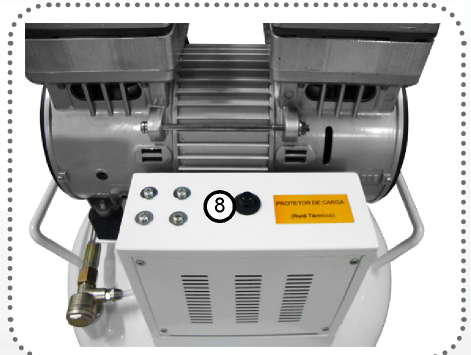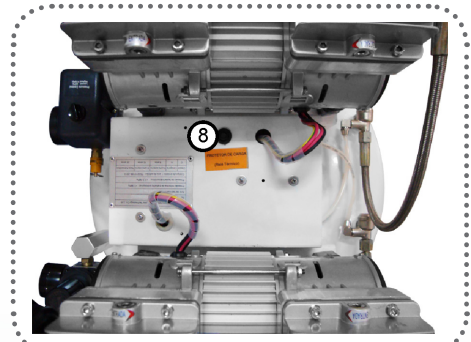The overload protector is designed to protect the equipment from overheating. If the overload protector (08) trips, wait for 5 minutes until the compressor cools down and reset it again; in the recurrence of the trip, turn off the equipment and see Dabi Atlante Technical Assistance.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

Cleaning

Never wet the compressor or any of their parts. The cleaning of outside surfaces should be performed weekly with a duster or soft cloth slightly moistened with water; the use of other products may damage the compressor finish or plastic parts. Be sure to thoroughly clean the head's upper part, responsible for its cooling. The accumulated dust affects the heat dissipation. If you find any problems in operation, follow the instructions below to check and fix the problem, and/or contact your representative.

UNFORESEEN EVENTS – SOLUTION OF PROBLEMS

Upon coming across any problem in operation, follow the instructions below to check and repair the problem, and/or get in touch with your representative.

Problem	Probable cause	Probable cause
- The compressor does not start.	- Power plug disconnected. - Main switch off. - Power outage in the network. - Overload protector tripped.	- Connect power plug into the outlet. - Connect main switch. - Wait for the regularization of electrical network. - Reset the protector according to the instructions (pag.17). In the recurrence of the protector trip, request the presence of a Dabi Atlante authorized technician.
- The compressor does not turn on and emits an unusual noise.	- Capacitor is burnt.	- Request the presence of a Dabi Atlante authorized technician.
- The compressor runs slowly.	- Electrical supply voltage lower than the specified.	- Connect the compressor to the correct network. - Wait for the regularization of network.
- The compressor does not reaches the maximum pressure, lesser efficiency, works for a short period of time.	- Pressure valve unregulated. - Air tank filled with water. - Air intake filter with excessive dirt.	- Request the presence of a Dabi Atlante authorized technician. - Drain the water from the tank (page 13). - Replace the air filter (page 17).
- The compressor motor does not turn off when it reaches maximum pressure.	- Pressure switch deregulated or defective.	- Request the presence of a Dabi Atlante authorized technician.

UNFORESEEN EVENTS – SOLUTION OF PROBLEMS

Problem	Probable cause	Probable cause
<p>- There is pressure drop of the reservoir.</p>	<p>- Leakage on the fittings.</p>	<p>- Usually this happens along the air line between the compressor and the equipment and not on the fittings of the compressor itself. Fill the air compressor as much as possible and with the help of a sponge and detergent, apply a little foam on the fittings, note where the leaks are and remake the fittings if problem persists; request the presence of a Dabi Atlante authorized technician.</p>

! If the problem is not listed in the column "Problem" or none of the solutions presented corrects it, request the presence of a Dabi Atlante authorized technician.

WARRANTY OF EQUIPMENT

This equipment is covered by the warranty terms and norms contained in the Warranty Certificate that accompany the product.

FINAL CONSIDERATIONS

Among the care you have to take with your equipment, the most important is regarding of the spare parts replacement.

To ensure the lifetime of your device, only replace original spare parts from Dabi Atlante. They have the assurance of the standards and technical specifications required by the Dabi Atlante representative.

We call your attention to our authorized resellers' chain. Only this chain will keep your equipment constantly new, because it has trained technical assistant and specific tools for the correct maintenance of your device.

Whenever you need, demand the presence of a Dabi Atlante' technician from the nearest resale, or ask through the Attendance Service Dabi Atlante: + 55 (16) 3512-1212.

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