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Retrosurgery Tips

Optional Configurations

Profi Neo and Profi Class only with ultrasound

Profi Neo and Profi Class only with bicarbonate jet

Profi Neo and Profi Class with additional air filter

Profi Neo and Profi Class without peristaltic pump

All components, parts and accessories described in this manual are to be solely used with **Dabi Atlante** products.

Contents

Legend	15
Control panel (Profi Neo)	15
Control panel (Profi Class)	15
Functions of control panel keys (Profi Class)	15
Installation	15
Additional procedures prior to equipment use/reutilization	15
Language selection (Profi Class)	15
Operation	16
Display messages (Profi Class)	16
Irrigant flow rate setting	18
Memory setting (Profi Class)	18
Adequate tip use of Endo/ Perio/ Retro ...	18
Correct file installation	18
Replacement of peristaltic pump	18
Cleaning	19
Sterilization	19
Water drainage	19
Maintenance	19
Recommendations, care and warnings	19
Procedures in case equipment functioning is altered	19
Sensitivity to predictable environmental conditions in normal working situations	20
Procedure in case of equipment disposal	20

Transportation and storage conditions	20
After installation	20
Preparation prior to delivery	20
Dabi Atlante Authorized Service Network	20
Symbols	20
Final recommendations	21
Equipment warranty	21
Reservoir replenishment	21
Technical features	21
Troubleshooting	22

Presentation

Dear Customer,

Welcome!

We are proud of your preference for our products. Your familiarization with the **Profi Line** will help maximize your equipment's performance and increase its durability and benefits. Therefore, before starting your equipment, be sure to read this Manual very attentively.

Important

All the information, illustrations and specifications in this manual are based on data existing at the time of its publication. Rights are reserved to make alterations at any time, both to the product and to this Manual, with out a previous notice.

Legend

- ① Sodium bicarbonate jet handpiece (fig. A and B)
- ② Sodium bicarbonate jet handpiece holder (fig. A and B)
- ③ Ultrasound handpiece (fig. A and B)
- ④ Ultrasound handpiece holder (fig. A and B)
- ⑤ Control panel (fig. A and B)
- ⑥ Foot control (fig. A, B and C)
- ⑦ Sodium bicarbonate reservoir (fig. C)
- ⑧ Fluid reservoir (fig. C and L)
- ⑨ Hose (fig. C and L)
- ⑩ Electric power cable (fig. C)
- ⑪ Compressed air input (fig. C)
- ⑫ Fluid reservoir cover (fig. C)
- ⑬ Air filter (fig. C)
- ⑭ On/Off main switch (fig. C)
- ⑮ Endo/ Perio/ Retro tips installation wrench (fig. F and G)
- ⑯ Handpiece protective cover (fig. G)
- ⑰ Dentistry Tip adapter (fig. H)
- ⑱ Tightening nut (fig. H)
- ⑲ Dentistry Tip (fig. H)
- ⑳ Endo L and Dentistry tips installation wrench (fig. K)
- ㉑ Reservoir fastening columns (fig. C e L)
- ㉒ Peristaltic pump (fig. C and L)
- ㉓ Peristaltic Pump arm (fig. C and L)
- ㉔ Hose connection (fig. L)

Control panel (Profi Neo)

- ① On/Off switch and selection of ultrasound and sodium bicarbonate jet functions
- ② Ultrasound power selection switch
- ③ Orange range indicator of recommended power limit for Endo Tips
- ④ Orange+brown range indicator of Perio Tips power limit
- ⑤ Peristaltic pump switch (on/off and fluid flow rate control)

Control panel (Profi Class)

- ① 'ON/OFF' key (fig. E)
- ② Key  (fig. E)
- ③ Key  (fig. E)
- ④ 'Peristaltic pump' key (fig. E) 
- ⑤ 'Bicarbonate jet' key (fig. E) 
- ⑥ 'Endo ultrasound' key (fig. E) 
- ⑦ 'Perio ultrasound' key (fig. E) 
- ⑧ Access to memory programming (fig. E)
- ⑨ Display (fig. E)
- ⑩ 'Dentistry' key (fig. E)

Functions of control panel keys (Profi Class)

Use the icons below to identify the control panel keys and their respective functions (fig. E):

-  ON/OFF key (fig. E, item 1).
-  'Perio ultrasound' key and power setting (fig. E, item 7).
-  'Endo ultrasound' key and power setting (fig. E, item 6).
-  'Sodium bicarbonate' jet function (fig. E, item 5).
-  'Peristaltic pump' function and irrigant flow rate setting (fig. E, item 4).
-  Power lowering and language selection (fig. E, item 2).
-  Power increasing and language selection (fig. E, item 3).
- 1 2 3 4** Access to memory programming (fig. E, item 8).
-  Dentistry Tip selection (fig. E, item 10)

Installation


This equipment must be unpacked and installed by an authorized Dabi Atlante technician, otherwise, warranty will be voided.

Additional procedures prior to equipment use/reutilization

Prior to equipment use/reutilization, follow the operation and cleaning instructions in this Manual.

Language selection (Profi Class)

Press key **4** and simultaneously turn the equipment on by setting the main switch (fig. C, item 14) to the 'ON' position.

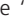Press key  or key  and when the display shows the desired language, press the "on" key .

Operation

Use of ultrasound

Attach the protective cover to the handpiece (fig. G, item 16).

Endo / Perio / Retro Tips

Insert the selected tip into the wrench (fig. F, item 15) and then screw it onto the handpiece (fig. G).

Dentistry Tips

Screw the adaptor (fig. H, item 17) onto the handpiece and tighten it firmly with the aid of an installation wrench (fig. K, item 20).

Screw the nut (fig. H, item 18) a little loosely and insert the desired Dentistry Tip, pushing it flush with the stop. The set must be like in (fig. I).

Tighten the nut with the installation wrench (fig. K, item 20).

To change the Tip, loosen the nut (fig. H, item 18), remove the Tip, insert the desired Tip and tighten the nut again.

 Never use the Dentistry Tips without the correct cooling.

Remove the reservoir cover (fig. C, item 12) and fill it up with your preferred fluid, without exceeding the maximum volume indication line (check section "Reservoir replenishment").

Turn the equipment on by setting the main switch (fig. C, item 14) to the 'ON' position.

Profi Neo - Set the selection switch (fig. D, item 1) to the "Ultrasound" function.

Profi Class - Press the ON/OFF key (fig. E, item 1) and then key  or .

Set the ultrasound power and the irrigant flow rate (check the corresponding sections in this Manual).

 To speed up such setting, we recommend the use of the memory keys in Profi Class.

Press the foot control to start the procedure (fig. A and B, item 6).

Upon finishing the procedure, release the foot control, place the handpiece into the holder and turn the equipment off.

Use of the sodium bicarbonate jet

Remove the sodium bicarbonate reservoir cover (fig. C, item 7) and if there are residues on the bottom, clean with the aspirator.

Rub the Dabi Atlante sodium bicarbonate packet for a while to homogenize it.

Pour one Dabi Atlante sodium bicarbonate packet inside the reservoir. Never pour more than the indicated by the volume level mark.

Replace the cover. Make sure the reservoir is tightly closed.

 Use only Dabi Atlante sodium bicarbonate, since other products may have varied grain size and uncontrolled moisture, which can clog the equipment.

Remove the fluid reservoir cover (fig. C, item 12) and fill it up with your preferred fluid (check section "Reservoir replenishment").

Turn the equipment on by setting the main switch (fig. C, item 14) to the 'ON' position.

Profi Neo – Set the selection switch (fig. D, item 1) to the bicarbonate jet function.

Profi Class - Press the ON/OFF key (fig. E, item 1) and then key .

Set the irrigant flow rate (check section "The irrigant flow rate setting").

 To speed up such setting, we recommend the use of the memory keys in Profi Class.

Press the foot control to start the procedure (fig. A and B, item 6).

Upon finishing the procedure, release the foot control, place the handpiece into the holder and turn the equipment off.


 In order to activate only the peristaltic pump in Profi Class (fig. E, item 4), disable the last function "sodium bicarbonate jet" or "ultrasound" by pressing the corresponding key.

Display messages (Profi Class)

1 - The DABI ATLANTE name will be displayed when the equipment is turned on, followed by a greeting:




DABI ATLANTE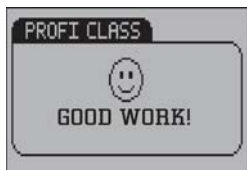

! The 'DABI ATLANTE' name will be displayed during a power outage and while the equipment is in standby mode.

2 - Observe the indicators below, which are used to display your Profi Class status: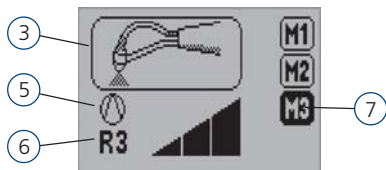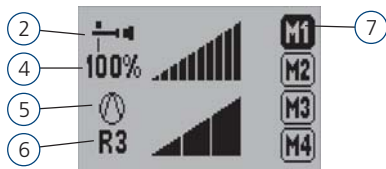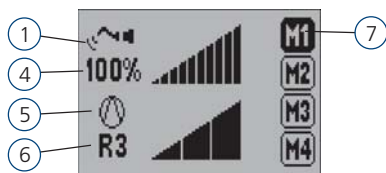

1 - Indicates the "Perio Ultrasound" function is actuated.

2 - Indicates the "Endo Ultrasound" function is actuated.

3 - Indicates the "Bicarbonate Jet" function is actuated.

! The "Sodium Bicarbonate Jet" function has no power indications, since it operates with stabilized air pressure.

4 - Indicates the effective power for the function. The power can be either increased or decreased by pressing keys **+** and **-**.

5 - Indicator that the "Peristaltic Pump" is actuated.

6 - Indicates the "Peristaltic Pump" flow rate level.

Flow rate can be adjusted by pressing key **⊞**.

There are 4 levels: R0 (off), R1, R2, and R3.

7 - Indicates the memory programming for the selected function.



8 - Indicates the "Dentistry" function is actuated.

9 - Indicates the selected "Dentistry Tip".

10 - Indicates the "Dentistry Tip" programming has been altered.

3 - Selection of Dentistry Tip:

Press the key **⏏** and the following screen will be displayed: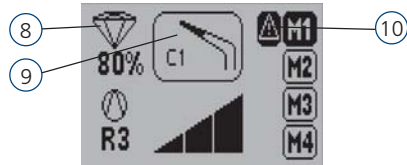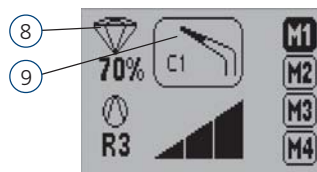

Press the key **⏏** until the desired Tip is selected.


The ultrasound power and the peristaltic pump flow rate are factory programmed.

In order to alter the programming, press **+** or **-** for power and **⊞** for the "Peristaltic Pump".

Both factory programming and new programming can be memorized in keys: **1 2 3 4**.

4 - Transducer cooling:

During the utilization of ultrasound with Tips that need no irrigation (water), the handpiece warms up. The Profi Class has a timekeeping system that automatically switches off the unit when necessary. At that moment, a warning will be displayed to wait for the unit to cool down.

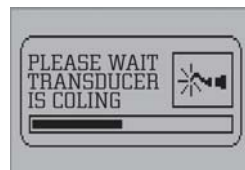5 - Replacement of the "Peristaltic Pump":


The system identifies pump wear and, if necessary, suggests its replacement by displaying the following warning. Press key **+** to clear the warning.

As long as the pump is not replaced, the Profi screen will display the warning  (item 11). (check section "Replacement of Peristaltic Pump").

To complete the replacement, press  and simultaneously turn the main switch on to inform that the pump has been replaced and to clear the warning  (item 11) being displayed.

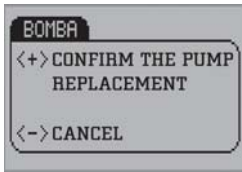To confirm the replacement, press .

Ultrasound power setting

Profi Neo

Select the desired power by means of the ultrasound power selection switch (fig. D, item 2).

-  Use the indicated power for the periodontics and endodontics functions.

Profi Class

Select the desired function (fig. E, item 6, 7 or 10) and press key  to decrease or key  to increase the power.

Irrigant flow rate setting

Profi Neo

Select fluid flow rate by means of the Peristaltic Pump switch (fig. D, item 5).

Profi Class


Press key  to adjust the speed.

-  The irrigant flow rate is related to the peristaltic pump speed.

Some procedures use ultrasound with no irrigation. In this case, the peristaltic pump can be deactivated by pressing key  until the display shows "R0". In the Bicarbonate Jet function, the pump cannot be deactivated.

Memory setting (Profi Class)

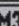Set the ultrasound power and the irrigant flow rate (check the corresponding sections in this Manual).

Keep the desired memory key pressed (   or ) until a *beep* is heard to save the selected setting.

-  After memory is set, the *display* will show M1, M2, M3 or M4.

Adequate *tip* use Endo / Perio / Retro

Use the gauge accompanying the scaler *kit* in order to check *tip* wear. Position the *tip* on the corresponding drawing and replace it in case wear exceeds 2 mm.

Correct file instalation

Manually screw the Endo L *tip* on the handpiece.

Softly tighten it using the Endo L *tip* installation key which is supplied with the ultrasound kit (fig. K, item 20).

Select the desired file, cut off its handle and insert the file into the Endo L *tip* orifice (fig. K).

Fix the file by tightening the Endo L *tip* nut using the installation key for that *tip* (fig. K, item 20).

Replacement of peristaltic pump

Remove the fluid reservoir cover (fig. C, item 12). Pull the hose (fig. L, item 9) until it detaches from the trough inside the reservoir. Pull the reservoir up until it detaches from the two body columns of the unit (fig. L, item 21).

Press the pump side rods (Fig. L, item 23) and pull the pump to detach it from the unit body.

Disconnect the hose (fig. L, item 24).

Connect the new pump hose and attach it to the unit body by pressing the side rods.

Insert the hose into the irrigant reservoir trough and attach it to the body fastening columns, making sure the hose is not kinked.

Cleaning

Clean the equipment surface with a clean soft cloth moistened in Aplic Odonto by Dabi Atlante or another product with similar chemical properties.

 The use of other chemical products is not recommended as it may damage the equipment.

To clean the fluid reservoir, remove it from the product (fig. L). Clean it and put it back.

 Remove the reservoir only when it is extremely necessary.

Sterilization

Sterilize the sodium bicarbonate jet handpieces, *tips*, *tip* installation keys and the handpiece protective cap by autoclave at up to 135 °C.

 Remove the sodium bicarbonate jet handpiece tip hose as it is not autoclavable.

Water drainage

After each day of use, place the equipment on a dry piece of cloth, incline it sideways at an angle of approximately 45°. Drain the air filter (fig. C, item 13) by pressing the depressurization pin until all the water has been eliminated.

 This procedure prevents clogging of the sodium bicarbonate jet tip.

Maintenance

Preventive maintenance

In order to reduce equipment faults and increased durability, call Dabi Atlante Authorized Technical Assistance and set a regular preventivemaintenance plan.

 **Profi Class:** The system identifies pump wear and, if necessary, suggests its replacement by displaying a warning.

Corrective maintenance

In case the equipment presents any faults that are not listed in "Troubleshooting", contact Dabi Atlante Authorized Technical Assistance.

 Do not open the equipment and/or try to repair it as such action may aggravate the problem or even generate further malfunctioning.

Circuit diagrams, list of parts, components, others

In case they are necessary, ask Dabi Atlante for electric diagrams and/or lists of parts, components or other information, which will be supplied by agreement.

Recommendations, care and warnings

In case the sodium bicarbonate jet tip becomes clogged, remove the hose and proceed with unclogging. (fig. J).

Sodium bicarbonate dehydrates lips. Use some type of moisturizer on the patient's lips prior to procedures.

Sodium bicarbonate use is contra-indicated for patients with renal complications or who are being submitted to hemodialysis.

The sodium bicarbonate powder spray may interfere with respiratory disorders. In such cases, medical advice is recommended.

The sodium bicarbonate jet handpiece must be loosened at least once a week to prevent thread locking.

 The use of ultrasound is contraindicated for patients and dentists wearing a pacemaker.

The scaler tips must be removed at least once a day to prevent thread locking.

 **Profi Neo:** When the Endo Tips are used with no cooling, the unit can be used for a maximum of 4 minutes, followed by an 8-minute waiting period, to prevent irreparable damage to the ultrasound handpiece.

Procedures in case equipment functioning is altered

In case the equipment presents any faults, firstly check whether they are listed under "Troubleshooting". In case the problem cannot be solved, turn off the equipment, unplug it from the power source outlet and call Dabi Atlante Authorized Technical Assistance.

Sensitivity to predictable environmental conditions in normal working situations

This equipment is not sensitive to magnetic, electric, electrostatic or pressure interferences as long as the cleaning, maintenance, transportation and operation recommendations found in this manual are observed.

Procedures in case of equipment disposal

When the equipment is put out of use, it must be disposed of at an appropriate site (according to local regulations), thus preventing its inadequate use and environmental contamination.

Transportation and storage conditions

The equipment must be transported and stored:

- Carefully in order to prevent falls and impact.
- By keeping the side indicated by the arrow facing upwards.
- By observing maximum piling limits as indicated on the packaging.
- Protected against moisture, rain, waters sprinkling and wet ground.

Transportation/storage environmental conditions	
Transportation and storage environmental temperature range	0°C to + 55°C
Transportation and storage relative humidity range	0% to 90% (non-condensing)
Atmospheric pressure range	500hPa to 1060 hPa (375 mmHg to 795 mmHg)

After installation

Keep the equipment protected from sunlight and rain.

Environmental operation conditions	
Environmental temperature functioning range	+10°C to +35°C
Environmental temperature range recommended by Dabi Atlante	+21°C to + 26°C
Functioning relative humidity range	30% to 75% (non-condensing)
Atmospheric pressure range	700 hPa to 1060 hPa (525 mmHg to 795 mmHg)

Environmental accommodation operation conditions (between operations)

Environmental temperature accommodation range	+5°C to +45°C
Environmental temperature range recommended by Dabi Atlante	+15°C to +30°C
Functioning relative humidity range	30% to 75% (non-condensing)
Atmospheric pressure range	700 hPa to 1060 hPa (525 mmHg to 795 mmHg)

Preparation prior to delivery

This equipment has been revised and approved according to the Dabi Atlante Quality Inspection and Control Program for better performance.

Dabi Atlante Authorized Service Network

Installation and all maintenance service for Dabi Atlante equipment must be given by Dabi Atlante Authorized Technical Assistance, otherwise warranty will be compromised.

Use the attached insert or access the web site www.dabi.com.br to find the nearest Dabi Atlante Authorized Technical Assistant.

Symbols

Use the icons below to identify the symbols on your equipment:

	Indicates the terminals for grounding protection.
	Attention: warns that accompanying documents must be consulted.
	Type-B equipment
	"Fragile" Located on the packaging side. It determines that equipment must be carefully transported in order to prevent falls or shocks.
	"Protect against humidity" Located on the packaging side. It determines that the equipment must be protected against any type of humidity during transportation and storage.

	<p>"This side up"</p> <p>Located on the packaging side. It determines that the packaging must be handled by keeping the side shown by the arrow facing upwards.</p>
	<p>"Maximum piling"</p> <p>Located on the packaging side. It determines the maximum number of boxes that can be piled up during transportation and storage.</p>
	<p>"Temperature limit"</p> <p>It determines the temperature limits at which the equipment must be stored or transported.</p>

Final recommendations

In order to ensure greater durability for your equipment, use only original Dabi Atlante replacement parts, which are in accordance with the technical standards and specifications required by the manufacturer.

Also, use only the services provided by the Dabi Atlante Authorized Technical Assistance Network as it counts on trained assistants and specific tools for the appropriate maintenance of your equipment.

Equipment Warranty

The "Warranty Terms" are found on the back of the certificate accompanying the product. Any alterations or concessions involving the warranty as well as pertaining conditions and authorizations, either oral or written, are definitely not permitted if not previously documented and approved of by Dabi Atlante.

Reservoir Replenishment

The table below shows possible concentrations. Others can be used on the dental surgeon's discretion.

Indicated anti-septic solutions	Suggested concentrations
Cetylpyridinium Chloride 1:2000 (Cepacol)	Pure
0,12% chlorhexidine (Periogard)	At 50% - 250 ml of the product and 250 ml of filtered water
Listerine	At 25% - 125 ml of the product and 375 ml of filtered water
Topical PVPI	At 50% - 250 ml of the product and 250 ml of filtered water
Sodium hypochlorite 3ppm*	0,6 ml of the product in 454 ml of filtered water

*Use only for endodontics.

Technical features

Item	Profi Neo and Profi Class	Item	Profi Neo and Profi Class
Height	215 mm	Voltage	99 to 242 V - 50/60Hz
Width	310 mm	Power	36 VA
Length	250 mm	Frequency	24 to 30kHz
Total weight	2,5 Kg	Protection fuse	2A
Operation device	Foot control	tip Amplitude	0,05 to 0,1 mm
Air consumption	10 l/min		
Compressed air primary pressure	60 to 80 PSI		

Technical responsibility by: Leonel Issa Halak - CREA-SP 0600256918

Troubleshooting

Faults	Causes	Solutions
Equipment will not start	Not plugged in the power source outlet	Plug it in
	Fuse is burned out	Replace fuse
	Power failure	Await normalization
	Circuit breaker is off	Turn it on
System is not pressurized	Compressor is off and/or valve is closed	Turn compressor on and/or open valve
<i>Tip</i> will not oscillate but LED goes on	<i>Tip</i> is not correctly fixed	Fix <i>tip</i> appropriately or try another <i>tip</i>
Water leakage in the handpiece	<i>Tip</i> is incorrectly screwed on	Screw <i>tip</i> on properly or replace O-ring
Sodium bicarbonate will not come out	Sodium bicarbonate jet handpiece terminal is not properly screwed on	Properly tighten the sodium bicarbonate jet handpiece terminal
	<i>Tip</i> is clogged	Unclog <i>tip</i>
	No sodium bicarbonate in the reservoir	Supply the reservoir
	Sodium bicarbonate excess in the	Remove excess
Water will not come out	No irrigating solution in the reservoir	Supply the reservoir
	Peristaltic pump is not enabled	Activate the peristaltic pump
Perception of electric shock on <i>tip</i>	No or ineffective grounding	Ground equipment adequately

Legal CE representative:

MPC International S.A - 26, Boulevard Royal - L-2449 - Luxembourg - Fone: +49 431-7028 219 - Fax: +49 431-7028 202

**ISO
13485**

By DNV OCS 0010

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OCS 0010

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DABI ATLANTE

+55 (16) 2512-1212

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Av. Pres. Castelo Branco, 2525 - 14095-000 - Ribeirão Preto - SP - Brasil