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ULTRALED

FOTOPOLIMERIZADOR
E CLAREADOR

CURING LIGHT AND
BLEACHER

Portugues.....	2
English.....	7
Español: www.dabi.com.br/esp	

DABI ATLANTE

Ultraled

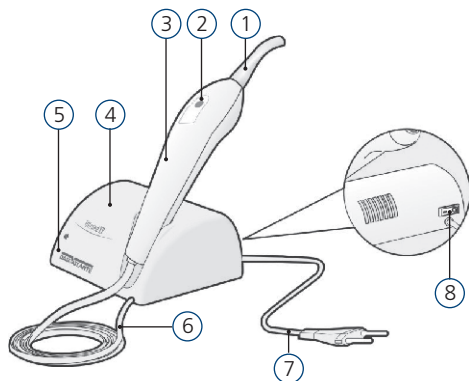
fig. A

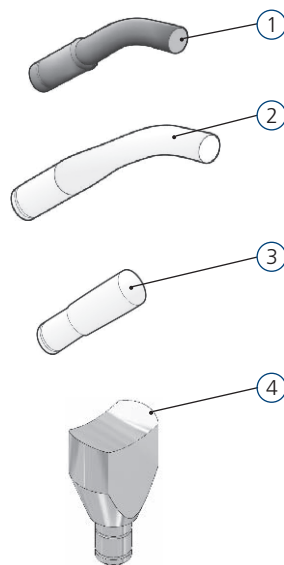
fig. B

Conteúdo

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Registro Anvisa nº: 10101130063

Contents

Legend	07
Ultraled	07
Light tips	07
Installation	07
Additional procedures prior to equipment use/reutilization	07
Operation	07
Cleaning	08
Sterilization	08
Maintenance	08
Recommendations	09
Procedures in case equipment functioning is altered	09
Sensitivity to predictable environmental conditions in normal working situations	09
Procedures in case of equipment disposal	09
Transportation and storage conditions	09
After installation	09
Preparation prior to delivery	09
Dabi Atlante Authorized Service Network ..	10
Symbols	10
Final recommendations	10
Equipment Warranty	10
Technical features	10
Troubleshooting	10

Presentation

Dear Client,

Welcome!

We are really proud of your preference for our product. Your familiarization with **Ultraled** curing light model will help maximize its performance and increase its durability and benefits. Therefore, before starting your equipment, be sure to read this Manual very attentively.

Important

All the information, illustrations and specifications contained in this manual are based on data existing at the time of its publication. We reserve the right to make any modifications deemed necessary both on the product and in this Manual without previous notice.

Legend

Ultraled (fig. A)

- 1 Acrylic curing light tip

- 2 Activation button
- 3 Handpiece
- 4 Base module
- 5 "Equipment-on" indicating LED
- 6 Extension cord
- 7 Power input cord
- 8 Master switch

Light Tips (fig. B)

- 1 Semi-guided curing light tip
- 2 Acrylic curing light tip
- 3 Acrylic light tip for one-tooth bleaching
- 4 Acrylic light tip for three-tooth bleaching


Installation

Light Curing (Photopolymerization)

Attach the acrylic curing light tip to the front orifice of the handpiece.

Plug the equipment in the power source outlet, and turn on its master switch.

The equipment will emit three short tones to indicate that it is on.

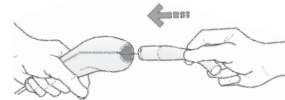Additional procedures prior to equipment use/reutilization

Prior to equipment use/reutilization, follow the operation and cleaning instructions in this Manual.

Operation

Move the handpiece into the patient's mouth and position the light tip at a maximum distance of 2mm from the material to be cured (photopolymerized).

The acrylic curing light is supplied with **Ultraled**. It is possible to use the semi-guided tip, which is sold separately. After the tip has been attached to the equipment, it should be cleaned with some gauze, since oiliness from the user's hands and other stains on the acrylic tip may affect light conduction.

Press the button to activate the LED, which will light up slowly. As soon as it is activated, the LED will remain on during the set time and a tone will be heard every 10 seconds.

When the programmed time has ended, the LED will go off. (refer to "Time setting").

 In case curing needs to be repeated, press the **Ultraled** activation button again. In order to interrupt the curing cycle, press the activation button again.

Bleaching

Attach the acrylic bleaching tip (fig. B, item 3 or 4) to the front orifice of the handpiece.

 The bleaching tips are sold separately. After the tip has been attached to the equipment, it should be cleaned with some gauze, since oiliness from the user's hands and other stains on the acrylic tip may affect light conduction.

Plug the equipment in the power source outlet, and turn on its master switch.

Follow the gel manufacturer's instructions as regards activation time and the distance between the tip and the gel.

Press the button to activate the LED, which will light up slowly. As soon as it is activated, the LED will remain on during the set time and a tone will be heard every 10 seconds.

When the programmed time has ended, the LED will go off. (refer to "Time setting").

 To interrupt the bleaching cycle, press the activation button again.

Time Setting

Activate the master switch to turn on the instrument and keep the handpiece activation button pressed until a long tone is heard. Release the activation button. Press it the number of times corresponding to the desired time setting. Each touch corresponds to 10 seconds. The maximum setting time is 100 seconds.

Intensity time adjustment program

Press the activation button on the handpiece to activate the LED. Then, without releasing the activation button, wait three seconds, and a long tone will be emitted. Still keeping the activation button pressed, wait until a second tone is emitted. Release the activation button and wait for a tone indicating the first setting phase, "Low Power". Then, press the activation button to set the application percentage. Each touch corresponds to 10% of the total pre-adjusted total functioning time, and the maximum will be 100% (in case the activation button is not pressed, the system will understand that no percentage has been set in this phase).

After the desired application percentage has been programmed, and if it is less than 100%, wait 2 seconds, and 2 tones will be emitted, indicating

the second setting phase, "Medium Power". Repeat the procedure used for the first phase. At this moment, only the remaining percentage can be programmed. "High Power" will correspond to the remaining percentage from the two first phases. Two seconds after the setting of the second phase has been concluded, 3 fast tones will be emitted, indicating setting completion. In case you wish to set only some of the phases, access the setting model as explained above, but do not press the activation button after each phase. The system will automatically shift to the next phase. When the activation button is not pressed during the setting of phase 1 and 2, the application will be set at 100% with "High Power".

Example: For an application time setting of 60 seconds and the phases respectively set at 10, 20 and 70%, the LED will remain on for 6 seconds at low power, for 12 seconds at medium power and for 42 seconds at high power.

Cleaning

Clean the equipment surface with a clean soft cloth moistened in Aplic Odonto by Dabi Atlante or another product with similar chemical properties

 The use of other chemical products is not recommended as it may damage the equipment. In case PVC film is used, make sure that the ventilation orifices (air inlet and outlet) are not clogged. The acrylic light tip should never be wrapped in any kind of material.

Sterilization

The acrylic curing light tip as well as the bleaching light tips must not be autoclaved.

Maintenance

Preventive maintenance

In order to reduce equipment faults and increase durability, call Dabi Atlante Authorized Technical Assistance and set a regular preventive maintenance plan.

Corrective maintenance

In case the equipment presents any faults that are not listed in "Troubleshooting", contact Dabi Atlante Authorized Technical Assistance.

 Do not open the equipment and/or try to repair it as such action may aggravate the problem or even generate further malfunctioning.

Circuit diagrams, list of parts, components, others

In case they are necessary, ask Dabi Atlante for electric diagrams and/or lists of parts, components

or other information, which will be supplied by agreement.

Recommendations

This equipment must be used only by properly trained personnel under supervision by qualified dental staff.

It is advised that the dental professional instruct his patient or other individuals who have access to the equipment as regards the caution to be taken during its handling.

This equipment is not appropriate for use in areas at risk of the explosion.

Prior to the first use and/or after long time work interruptions, such as vacation periods, clean and disinfect the equipment.

Do not directly expose the equipment to sunlight as that will cause early wear.

Prior to connecting the charger base to the power supply, make sure that the main voltage is in accordance with the charger's voltage range.

Do not look at the light without the eye protector as it may be harmful.

Do not place the unit near flammable items, and do not place the light in contact with flammable solutions.

In order to prevent electric shock, do not insert any other parts into the unit.

Procedures in case equipment functioning is altered

In case the equipment presents any faults, firstly check whether they are listed under "Troubleshooting". In case the problem cannot be solved, turn off the equipment, unplug it from the power source outlet and call Dabi Atlante Authorized Technical Assistance.

Sensitivity to predictable environmental conditions in normal working situations

This equipment is not sensitive to magnetic, electric, electrostatic or pressure interferences as long as the cleaning, maintenance, transportation and operation recommendations found in this manual are observed.

Procedures in case of equipment disposal

When the equipment is put out of use, it must be disposed of at an appropriate site (according to local regulations), thus preventing its inadequate use and environmental contamination.

Transportation and storage conditions

The equipment must be transported and stored:

- Carefully in order to prevent falls and impact.
- By keeping the side indicated by arrow facing upwards.
- By observing maximum piling limits as indicated on the packaging.
- Protected against humidity, rain, water sprinkling and wet ground.

Transportation /storage environmental conditions

Transportation and storage environmental temperature range	0°C to + 55°C
Transportation and storage relative humidity range	0 % to 90 % (non-condensing)
Atmospheric pressure range	500hPa to 1060 hPa (375 mmHg to 795 mmHg)

After installation

Keep the equipment protected from sunlight and rain.

Environmental operation conditions

Environmental temperature functioning range	+10°C to +35°C
Environmental temperature range recommended by Dabi Atlante	+21°C to + 26°C
Functioning relative humidity range	30% to 75% (non-condensing)
Atmospheric pressure range	700 hPa to 1060 hPa (525 mmHg to 795 mmHg)

Environmental accommodation operation conditions (between operations)

Environmental temperature accommodation range	+5°C to +45°C
Environmental temperature range recommended by Dabi Atlante	+15°C to +30°C
Functioning relative humidity range	30% to 75% (non-condensing)
Atmospheric pressure range	700 hPa to 1060 hPa (525 mmHg to 795 mmHg)

Preparation prior to delivery

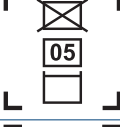
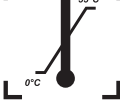
This equipment has been revised and approved according to the Dabi Atlante Quality Inspection and Control Program for better performance.

Dabi Atlante Authorized Service Network

Maintenance service for Dabi Atlante equipment must be given by Dabi Atlante Authorized Technical Assistance, otherwise warranty will be compromised. Use the attached insert or access the website www.dabi.com.br to find the nearest Dabi Atlante Authorized Technical Assistant.

Symbols

Use the icons below to identify the symbols on your equipment:

	"Fragile" – Located on the packaging side. It determines that equipment must be carefully transported in order to prevent falls or shocks.
	"Protect against humidity" – Located on the packaging side. It determines that the equipment must be protected against any type of humidity during transportation and storage.
	"This side up" – Located on the packaging side. It determines that the packaging must be handled by keeping the side shown by the arrow facing upwards.
	"Maximum piling" – Located on the packaging side. It determines the maximum number of boxes that can be piled up during transportation and storage.
	"Temperature limit" – It determines the temperature limits at which the equipment must be stored or transported.

Final recommendations

In order to ensure greater durability for your equipment, use only original Dabi Atlante replacement parts, which are in accordance with the technical standards and specifications required by the manufacturer. Also, use only the services provided by the Dabi Atlante Authorized Technical Assistance Network as it counts on trained assistants and specific tools for the appropriate maintenance of your equipment.

Equipment Warranty

This equipment's warranty covers manufacturing faults for a period of 12 months starting on the date of its sales receipt issuance in the name of its first owner.

Reasons for warranty voidance:

- Installation and/or operation in disagreement with the instructions in this manual.
- Falls.

Items that are not covered by warranty:

- Technicians' travel expenses and equipment transportation expenses for technical assistance.
- Damage to external finishing and/or accessories that has not been caused by the manufacturer.

Note

In case technical assistance is required during the warranty coverage period, the sales receipt must always be presented to the technician providing such service.

Technical features

Specification	Ultraled
Voltage	100-240Vac
Frequency	50/60Hz
Power	10VA
Light Source	
Light wavelength	460-490nm
Light intensity	>500mW/cm ² *
Light guiding tip	∅ 10 mm angle: 60°

*In the operation conditions specified on this Manual

Troubleshooting

Fault	Cause	Solution
Equipment will not start	There is no power supply	Await for power supply normalization or call for technical assistance
Equipment will not cure	LED is burned out	Replace the head
	Obstruction of the light guiding tip	Replace and clean it

Technical responsibility by: Leonel Issa Halak – CREA-SP 0600256918

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